

**(((BIRTH
ALARM)))**

the original product

Birth Alarm Premium

EN Manual



Thank you for purchasing the Birth Alarm Premium. This is an extremely reliable reporting system for foal births. We advise you to read this manual carefully before use; proper assembly and operation are essential for correct operation.

Contents


1.	Introduction.....	4
2.	Contents of the packaging	4
3.	Preparing for use.....	5
3.1	The first time	5
3.2	Switching on/off	6
4	Components of Birth Alarm Premium.....	7
4.1	Installing/replacing batteries.....	7
4.2	Housing of the Birth Alarm Premium	8
4.3	Installing the SIM card.....	9
5	The lights (Leds) of the Birth Alarm Premium.....	10
5.1	The status lights	10
5.2	De LTE lamps	11
6	Installing a new SIM card	12
6.1	SIM card choice	12
6.2	Preparing a SIM card (general).....	12
6.2.1	Prepaid SIM cards and prepaid call credit	12
6.2.2	Subscription card or Internet/data cards.....	12
6.3	Preparing the SIM card.....	13
7	Programming telephone numbers.....	14
7.1	Programming mode	14
7.2	Programming telephone numbers.....	14
7.2.1	Programming by dialling	14
7.2.2	Programming by SMS.....	15
7.2.3	Direct programming	16
8.	Programming other settings	17
8.1	Sleep mode.....	17
8.2	Angle setting.....	17
8.3	Number of calls and SMS moment	18
8.3.1	Change number of call rounds	18
8.3.2	SM Setting the SMS moment	18
8.4	Reset to factory default settings.....	19
9.	The SMS status message	20
10.	Testing	21
11	Fitting the Birth Alarm onto the mare	22
11.2	Fitting the Birth Alarm beneath the halter	23
11.3	Fitting the Birth Alarm into an anti-roll girth	23
12	Summary and modes of use.....	24
13	Technical data	25

1. Introduction

When your mare is giving birth you would like to be present so that you can intervene in case of any complications or warn a veterinarian. Horses and ponies have the tendency of lying stretched out on their left or right side during delivery. This gives them more space during contractions. The Birth Alarm Premium makes good use of this tendency. By simply fitting the Birth Alarm Premium beneath the mare's halter (or into an anti-roll girth), it will warn you when the delivery starts.

2. Contents of the packaging

1. **The Birth Alarm Premium birth detector.**
2. **The special holder.** (This is where the Birth Alarm Premium is inserted.) The holder can be fitted beneath the mare's halter or in the ring of the anti-rolling girth.
3. **Cable ties.** To get started, we have already supplied a number of cable ties (also called "Tie-wraps"). These cable ties are for single use, but they are also available at any DIY store.
4. **The special screwdriver.** This is to unscrew your Birth Alarm Premium (e.g. when the batteries need to be replaced or when you need to remove the SIM card). But there is also a strong magnet on the back of this screwdriver. It allows you to operate the Birth Alarm Premium.
5. **USB cable.** You will need this - or a similar cable - if you want to update the system software.
6. **SIM card adapter set.** In this set you will find a number of holders with which you can adjust the size of your SIM card. You don't necessarily need this, but it can be helpful if you want to set the SIM card with your own phone and your own phone accepts a different size SIM card.
7. **Documentation.** You will find 2 types of documentation with your Birth Alarm Premium. Namely:
 - a. **QuickStart Guide** (short and concise manual)
 - b. **TroubleShooter** (You can use this checklist if you have problems using the Birth Alarm Premium)

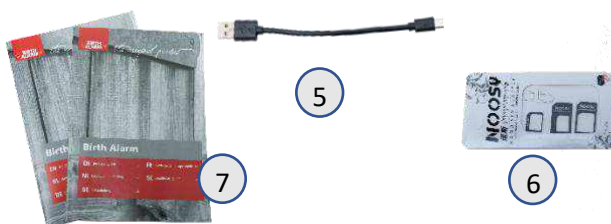

3. Preparing for use

3.1 The first time

If you wish to use Birth Alarm Premium for the first time, you must perform these steps once:

- Unscrew the Birth Alarm Premium and remove the red label which is placed against the pole of one of the batteries. The batteries now make contact with the Birth Alarm Premium.
- Install the SIM card. SIM card not included, but can be easily purchased at a mobile phone provider or local supermarket.
- The chapter "Installing a new SIM card" (**see 6**) explains how to set up and install a newly purchased SIM card.
- Once you have completed these steps with your Birth Alarm Premium, you can close it again and continue with the settings. When screwing in, ensure that the light conductors are placed above the status lights (**see 4.2**)



Unscrewing



Removing the Label



If you have good reception with your mobile phone in the stable or field where your mare is, we recommend that you purchase a SIM card from the same provider.



There may be special regulations in your country regarding the use of SIM cards in automatic mobile systems (M2M). Ask your mobile phone provider or mobile phone specialist whether this is the case.

3.2 Switching on/off

You can switch the Birth Alarm Premium on/off and operate it with the back of the special screwdriver supplied. On one of the long sides of the casing you will find 2 status lights.

To switch the Birth Alarm Premium **ON**, proceed as follows:

- Place the back of the special screwdriver against one of the status lights. (it does not matter which of the two)
- The Birth Alarm Premium will now switch on. This is indicated by both status lights illuminating green.
- You can now remove the back of the screwdriver.
- Your Birth Alarm Premium is now switched on. The status lights will briefly illuminate green every ~5 seconds to indicate that the Birth Alarm Premium is switched on and that the status is correct.



To switch the Birth Alarm Premium **OFF**, proceed as follows:


- Place the back of the special screwdriver against one of the status lights. (it does not matter which of the two)
- The Birth Alarm Premium will now switch off.
- You can now remove the back of the screwdriver.
- Your Birth Alarm Premium is now switched off. The status lights will stay off.



4 De binnenkant van de Birth Alarm Premium

Below is an overview of the system components.

1. Batteries.
2. The 2 status lights
3. SIM card slot
4. USB-connector (for software update)
5. LTE status led
 - 5.1 "NET" – led
 - 5.2 "RING" – led
 - 5.3 "LTE" – led

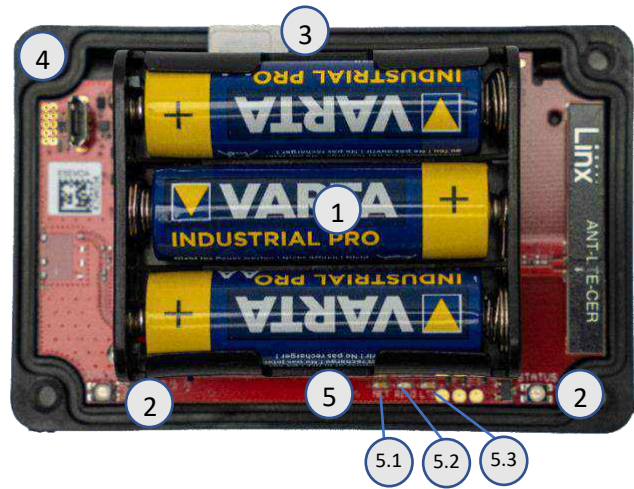

4.1 Installing/replacing batteries

When replacing the batteries for your Birth Alarm Premium you should always use alkaline batteries (preferably "Duracell", "Panasonic", or "Varta"). The use of rechargeable batteries is not recommended. Always replace all 3 batteries at the same time. If you cannot remove the batteries with your fingers, you can also use a small screwdriver, as long as it is done with care.

The batteries must always be placed with their negative pole on the spring.



Always use new batteries at the beginning of the foaling season!



If you will not be using Birth Alarm Premium for a longer period of time, such as after the foaling season, you should remove the batteries to prevent them from leaking and causing damage to your Birth Alarm Premium.

4.2 Housing of the Birth Alarm Premium

The housing of the Birth Alarm Premium can only be screwed down in one way. The light guides on the long side of the lid must be on top of the status lights when the lid is screwed down.



4.3 Inserting the SIM Card


Your Birth Alarm Premium works with a micro-sim. Modern SIM cards come in a break-out frame from which you can click out the micro SIM format. Please note that the micro-sim format is usually the second smallest format at 12 x 15 mm.



The SIM card must be inserted into the Birth Alarm with the chip at the bottom. Also pay attention to the slanted edge of the SIM card. It must point forward when you insert the SIM card.

Push the SIM card into the slot until you hear a "click". Then release the SIM card again. The SIM card is now in the slot and the Birth Alarm Premium is ready to be used. When you want to remove the SIM card from the slot, gently push against the SIM card until you hear another "click". If you now release the SIM card again, the SIM card will come forward so that you can easily remove it.

(For more information about the SIM card see chapter 6)

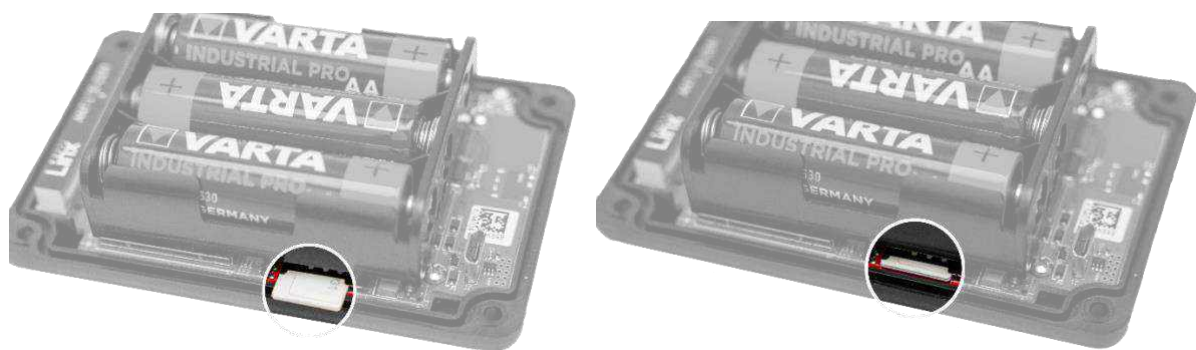


5 Birth Alarm Premium (Led) lights

To view the status of your Birth Alarm Premium, there are 2 status lights on your Birth Alarm Premium and 3 small lights on the inside that only relate to the internal LTE module.

5.1 The status lights

The status lights are on the long side of the casing. These lights both behave exactly the same. The double version is only meant to indicate the status of your mare both left and right at all times.

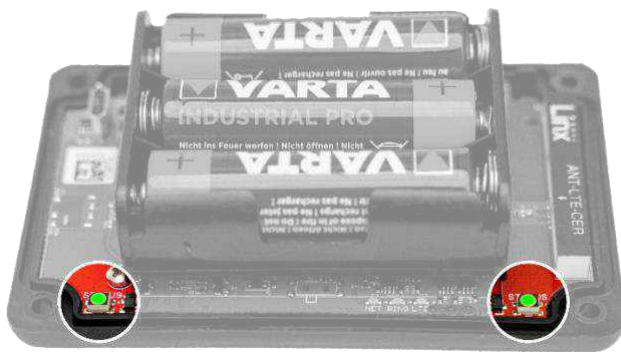

In the table below you will find the possible colour patterns that the Birth Alarm Premium can display.

Colour / pattern	Status
● ●	● Stand-by, battery okay
●● ●●	●● Stand-by, battery okay, sleepmode enabled
● ●	● Stand-by, battery weak
● ●	● LTE/GSM active, connecting
● ● ● ●	● ● LTE/GSM active, network reception insufficient
● ● ● ●	● ● LTE/GSM active, network reception moderate
● ● ● ●	● ● LTE/GSM active, network reception good

5.2 The LTE lamps

The LTE lamps are only visible when opening the Birth Alarm Premium. Normally, you don't need to read these lamps, but it may come in handy if you are experiencing problems with the mobile connection and contact customer service for this.



In the table below you will find the possible flashing patterns and their meaning:

Name	Pattern	Status
NET		Module connects to the network
NET		Module is connected to the network
NET		Module is calling
RING		Module receives a call
LTE		Module is switched on

6 Installing a new SIM card

The SIM card is not included. You will have to purchase it yourself.

6.1 SIM card choice

Buy a prepaid SIM card or possibly a SIM card with a cheap Sim-Only or data + voice contract at your mobile specialist shop. Preferably choose a provider that you know offers good coverage at the location where you want to use Birth Alarm Premium. Usually, a prepaid card is the easiest to use. However, it is possible that the regulations for SIM card use in automatic mobile systems (M2M) in your country do not allow the use of a regular mobile number. Please check whether this is the case.



In the Netherlands, for example, it is not permitted to use a prepaid card from a regular mobile number for an automated calling system. Go to your telecom provider for a suitable contract or visit www.birthalarm.com for a suitable SIM card solution.

6.2 Preparing a SIM card (general)

When you have purchased a SIM card, you still need to activate it. This goes as follows:

- Insert the newly purchased SIM card into your device.
- If your device asks for a PIN code after switching on, enter it. The PIN is mentioned in the booklet or in the letter that comes with your SIM card. Incidentally, in most cases the PIN code is "0000". **Then you need to deactivate this pin protection on the SIM card. Your phone is not allowed to ask for a PIN code when you switch it on with the SIM card. Refer to the user manual of your telephone to find out how this works.**

6.2.1 Prepaid SIM cards and prepaid call credit

If you are using a prepaid SIM card, it is wise to keep an eye on your credit balance. With normal use, the call and SMS costs of your Birth Alarm Premium are minimal, but still check the prepaid SIM card credit regularly. This is done by inserting the SIM card into your device and calling your provider's call credit/ top-up service line. Make sure that there is at least £10 on the card, and that the credit remains valid for a sufficient period of time. If the prepaid credit is insufficient, you will need to top it up via a credit card, which is often available at supermarkets or petrol stations, or via a top-up code that you can buy online.

6.2.2 Subscription card or Internet/data cards

If you use a subscription card/data card, make sure that your provider enables voice and SMS functionality on your card. In most cases, this is not activated automatically for data cards.

6.3 Preparing the SIM card.

Once you have set up your SIM card you will need to do the following:

- Call someone at least once. It may be that the provider sends a welcome message, wait for this to happen, and wait until the chosen number rings.
- Now check whether the voicemail of the SIM card is switched off. This is done by calling the number of the SIM card with another device. If you are connected to a voicemail service instantly or after some time, the SIM card's voicemail is switched on. In this case, you should disable voicemail.
- Usually, you can deactivate the voicemail service by calling the provider's customer service or by entering and calling a specific deactivation code. How to deactivate the voicemail is different for each provider and is mentioned in the manual of your SIM card.
- Now remove the SIM card from your device and insert it into the Birth Alarm Premium (see 4.3).



Never program the SIM card's own number on M1 or any of the SIM card's other memory locations!



7 Programming telephone numbers

7.1 Programming mode

To programme the Birth Alarm Premium directly, proceed as follows:

Place the back of the screwdriver against one of the status lights on the long side of the casing **for a few seconds**. While doing this, the status lights will first illuminate green 3 times and then blue. When the status lights illuminate blue you can remove the screwdriver. The Birth Alarm Premium is now in programming mode.

7.2 Programming telephone numbers

Programming the telephone numbers can be done in 3 ways. Namely, by calling, by SMS or by direct programming. All possibilities are described in detail below. **However, you only need to choose 1 method.** "Calling" is the most popular. The 4 numbers that the Birth Alarm Premium can call have the names "M1", "M2", "M3" and "M4".

7.2.1 Programming by dialing

- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately flashes blue once- green once.
- Now you call the Birth Alarm Premium. Preferably with your mobile phone.
- The Birth Alarm Premium will now receive your call and programme your telephone number into its memory.
- In confirmation of this, the Birth Alarm Premium will now call you back. Just let your phone ring, don't answer and don't refuse the call.



Programming is now complete. The Birth Alarm Premium now knows your number and will call you when the mare is giving birth. You can now wait 2-3 minutes. The Birth Alarm Premium will automatically exit the programming mode and is ready to be used.



When you have finished programming wait 2-3 minutes, the Birth Alarm Premium will automatically exit the programming mode and return to normal. The Birth Alarm Premium is then simply switched on and ready to be used again. You can leave the programming mode immediately by again placing the back of the screwdriver against one of the status lights.

If you would like to pre-program several more numbers, please follow the steps below.

- Wait until the Birth Alarm Premium flashes blue once, green **twice**.
- Now you call the Birth Alarm Premium with the device on which you want to be called on in **2nd** place (this may also be a landline telephone connection).
- The Birth Alarm Premium will now receive your call and programme your telephone number into its memory.
- In confirmation of this, the Birth Alarm Premium will now call you back. Just let your phone ring and don't react to it.

- After a few seconds, the Birth Alarm Premium will flash blue once and green **three times** to indicate that you can now programme the **3rd** number (and later the 4th number) in the same way.

Light pattern status-LED	Status
	The module is ready for the first call
	The module is ready for the second call
	The module is ready for the third call
	The module is ready for the fourth call



The number of green flashes on the Birth Alarm Premium indicates which number is programmed. For example, blue once and green **four times** mean that you programme the **4th** number.

7.2.2 Programming by SMS

You can also programme the Birth Alarm Premium through SMS. Please proceed as follows: Send an SMS message on your mobile phone saying: "**M1:**" (Caution! Use capital **M**) immediately followed by the desired first number. The table below shows an example of an SMS in your country.

Country	Country code	Example of your mobile number:	Voorbeeld van de op te stellen SMS:
Netherlands (NL)	+31	0 6 12345678	M1:+31612345678
Belgium (BE)	+32	0 491 123456	M1:+32491123456
France(FR)	+33	0 6 12345678	M1:+33612345678
Switzerland (CH)	+41	0 176 12345678	M1:+4117612345678
Austria (AT)	+43	0 664 1234567	M1:+436641234567
England (UK)	+44	0 7788 123456	M1:+447788123456
Sweden (CH)	+46	0 707 123456	M1:+46707123456
Germany (DE)	+49	0 171 1234567	M1:+491711234567

(Note: use capital "M" and type everything together without spaces)



So, as you can see in the table, you actually program the country code instead of the very first "0" you are normally used to when calling a mobile number.

- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*) . Wait until the status light alternately flashes blue once- green once.

- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. This contains the number you have programmed behind M1:
- Of course, you can also programme the numbers of M2, M3 and M4 in the same way. You can also program several numbers in one SMS simultaneously, but start a new line in your SMS for each number.




If you want to delete a number via SMS, simply send the corresponding memory location without a number behind it.



It is strongly recommended to programme a mobile number as the first number (M1). This is how the Birth Alarm Premium can send you SMS messages and receive SMS messages from you



You can also programme the numbers via SMS while you test the birth detector. (see 10) While the NET light on the Birth Alarm Premium is flashing quietly (long green with short interval), you can send an SMS with telephone number(s) or new settings, and Birth Alarm Premium can process them.

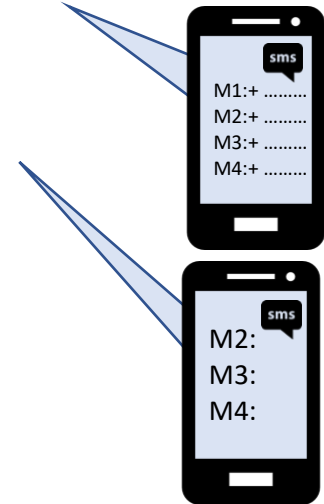


7.2.3 Direct programming

There is always the option of programming Birth Alarm Premium by temporarily inserting the Birth Alarm Premium SIM card into your own or another mobile phone and programming the numbers on the SIM card via this phone. This is usually a more complex way. To programme the SIM card directly, proceed as follows:

- Now insert the SIM card into another mobile phone.
- Switch on the mobile phone.



If you do not succeed in programming the numbers on the SIM card memory places 11, 12, 13 and 14 via your mobile phone, you can also insert the SIM card into the Birth Alarm Premium and set it to programming mode. The Birth Alarm Premium will notice that the SIM card is not configured and will programme the memories in the right places on the SIM card so that you can use them.

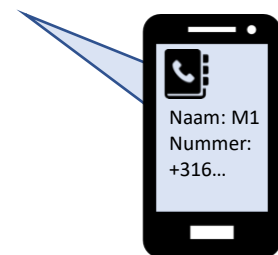



8. Programming other settings

The Birth Alarm Premium comes with standard settings from the factory. Usually, these are good for the majority of all mares (horses/ponies). But if you wish, you can change these default setting

8.1 Sleep mode

You use the "sleep mode" of the Birth Alarm Premium if your peri-parturient mare rests fully stretched (sleeps). The sleep mode requires a certain adjustment in the operation of the Birth Alarm Premium. This adjustment ensures that an alarm as a result of the mare resting fully stretched is prevented. When you use the sleep mode, the Birth Alarm Premium will check if your mare is lying stretched and is restless as well, as is the case with contractions. You can **enable** the sleep mode as follows:

- Create an SMS message on your mobile phone saying: "**SL:1**" (Note: use the capitals **SL**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. It now says "SL:1" on the last line as a sign that the sleep mode is switched on.

Similarly, you can always **switch off** the sleep mode:

- Create an SMS message on your mobile phone saying: "**SL:0**" (Note: use the capitals **SL**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. It now says "SL:0" on the last line as a sign that the sleep mode is switched off. The Birth Alarm Premium now works normally again.

8.2 Angle setting

The Birth Alarm Premium responds to the mare lying down with extended legs. In other words, it "measures" the extent to which your mare lies on her side. On average, an angle greater than 65° may be considered "abnormal" and the Birth Alarm Premium will act on it. The Birth Alarm Premium is set at 65° by default. However, you can raise this 65° threshold yourself, so that the Birth Alarm Premium comes into action later, i.e. becomes less sensitive. To change the angle setting (e.g. to 70°) use the following steps:

- Create an SMS message on your mobile phone saying: "**AN:70**" (Note: use capital **A**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. This now shows "A1:70" on the third last line as a sign that the Birth Alarm Premium is now active at a lying angle of 70°.

You can set the angle between 60° and 75°.



for example, you receive an alarm in the status SMS that the mare is just lying at 65°, but nothing is wrong, you can adjust the detection angle for this mare slightly upwards, for example to 70°. Do not go beyond 75°!

8.3 Number of calls and SMS moment

You can set the number of times the Birth Alarm Premium calls you and also the moment it sends you an SMS. By default, the Birth Alarm Premium will make 2 rounds of calls in the event of an alarm. That means it calls the numbers M1, M2, M3 and M4, which have been programmed successively and then calls all the numbers again. After this, the Birth Alarm Premium will automatically stop calling. After the Birth Alarm Premium has called the number M1 for the 2nd time, it will also send a status SMS to M1. If you want the Birth Alarm Premium to call you more often, or if you want the SMS to be sent earlier or later, you can change this yourself.

8.3.1 Change number of call rounds

To adjust the number of call rounds - to 5 for example - proceed as follows:

- Create an SMS message on your mobile phone saying: "**CL:5**" (Note: use the capitals **CL**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. It now says "CL:5" on the penultimate line, indicating that the Birth Alarm Premium will now call the list of numbers 5 times in the event of an alarm before it stops automatically.

8.3.2 SM Setting the SMS moment

You can set whether and when you want to receive an SMS status message.

8.3.2.1 No SMS

- If you do NOT wish to receive SMS messages from Birth Alarm Premium, please carry out the following steps:
- Create an SMS message on your mobile phone saying: "**MS:0**" (Note: use capitals **MS**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just set up to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. This now contains the penultimate line but one

8.3.2.2 Always SMS

If you wish to receive an SMS message from Birth Alarm Premium for every alarm, please carry out the following steps:

- Create an SMS message on your mobile phone saying: "**MS:1**" (Note: use the capitals **MS**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just created to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. It now reads "**MS:1**" on the penultimate line, indicating that the Birth Alarm Premium will send you an SMS status message after every call.


8.3.2.3 SMS after 2nd call

If you only want to receive an SMS message from Birth Alarm Premium after the 2nd call (Birth Alarm Premium is set by default this way), carry out the following steps:

- Create an SMS message on your mobile phone saying: "**MS:2**" (Note: use the capitals **MS**)
- Set the Birth Alarm Premium to "Programming mode" (*see 7.1*). Wait until the status light alternately illuminates blue once- green once.
- Now send the SMS you have just created to the Birth Alarm Premium mobile number.
- After a short period of time you will receive a status SMS from the Birth Alarm Premium as confirmation. It now reads "**MS:2**" on the penultimate line, indicating that the Birth Alarm Premium will only send you an SMS status message after calling twice.

8.4 Reset to factory default settings

You can reset all settings to factory settings by texting the word "RESET" to your Birth Alarm Premium. This restores all settings to default values. The programmed telephone numbers are not deleted now.

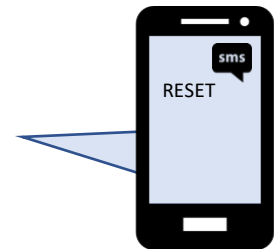

9. The SMS status message

Birth Alarm Premium will send you an SMS status message if you have changed a setting or telephone number on Birth Alarm Premium or if you receive a birth notification. This SMS status message provides you with information about the current settings and programmed telephone numbers on Birth Alarm Premium. It also gives you some measured values regarding the mare and Birth Alarm Premium itself.

	SMS-Message:	Meaning:
Measured values	Battery 60 %	Remaining battery capacity
	Signal 51%	Signal Strength
	Angle: 65	Measured lying angle in degrees
	SMS: 12	Number of SMS messages already sent: 12
Programme settings	M1: +31612345678	This is the first number to be dialed
	M2: +31644332211	This is the second number to be dialed
	M3:	Third number (not used here)
	M4:	Third number (not used here)
	AN:65	Detection angle is set to 65 degrees
	MS:2 CL:2	SMS after 2nd call. Total number of call rounds is 2
	SL:0	Sleep mode is switched off (0)

- *When you see that the battery percentage drops below 20%, the batteries must be replaced!*
- *When you see that the signal strength is sometimes lower than 20% we advise you to choose another provider that has better coverage at the place where you use the Birth Alarm Premium.*
- *The SMS counter gives you insight into the consumption of call credit or call minutes.*



"M1" stands for "Memory 1".

"AN" stands for "Angle".

"MS" stands for "Message".

"CL" stands for "Calls".

"SL" stands for "Sleepmode".

10. Testing

Before you start using Birth Alarm Premium, it is important to test it to ensure that it works properly. You can easily do the testing yourself.

- Preferably take the Birth Alarm Premium to where your mare is located.
- Take the Birth Alarm Premium in your hands and switch it on so that the status light flashes green once every 4 seconds.
- Now turn the Birth Alarm Premium 90 degrees so that one of the narrow sides is facing upwards. (*Attention: When using the sleep mode, the Birth Alarm Premium should also be horizontal again after 15 seconds*)
- The status light will now flash orange.
- After a few seconds, the status light turns blue, and a connection is built on it.
- Within a short time the Birth Alarm Premium will now call the number programmed under "M1".
- You do not answer or reject the call via the red button. So you just let the phone ring.
- The Birth Alarm Premium stops ringing automatically after a few seconds.

Now you can stop the Birth Alarm Premium by simply calling it. The Birth Alarm Premium will reject your call, but now you have stopped the Birth Alarm Premium. The Birth Alarm Premium will then not call any further numbers unless it is tilted again after being below the critical angle for a few minutes.

If the test worked, Birth Alarm Premium is ready to use on your mare.



If you still experience problems, you can use the "troubleshooter" provided.



You can programme the mobile number of the Birth Alarm Premium into your own mobile phone with the name "Mare" or "Birth". You will then be able to see in advance in the screen of your mobile phone when the Birth Alarm Premium calls you, so you do not have to answer the call.



When you receive a call from Birth Alarm Premium - whether it is a birth announcement or during programming - never answer a call from Birth Alarm Premium. When you pick up, you are basically having "a conversation" with the Birth Alarm Premium and that costs unnecessary money! Don't push a call away either (pressing the red button) because in that case, the Birth Alarm Premium may be forwarded to your voicemail.

11 Fitting the Birth Alarm onto the mare

To use the Birth Alarm Premium in a girth or underneath the halter, it must be attached to the holder supplied as follows:

- Carefully slide the Birth Alarm Premium into the holder as shown in the picture below.
- Make sure that both status lights are visible.
- Then close the Flap with the supplied closing clip.



In the example below, the status lights are placed in the direction of the locking clip, but you may also place the casing the other way around. This does not affect the operation.




11.2 Fitting the Birth Alarm beneath the halter

If you want to use the Birth Alarm Premium beneath a halter, mount it as shown below using 3 tie-wraps. Make sure that the halter is slightly tight so that the Birth Alarm Premium hangs straight under the mare's jaw.



11.3 Fitting the Birth Alarm into an anti-roll girth

If you are using the Birth Alarm Premium in an anti-roll girth, you must mount the holder with the Birth Alarm Premium in accordance with the illustration below. You use 6 tie-wraps for securing it into an anti-roll girth. Make sure that the detector is firmly placed horizontally in relation to the back of the mare.

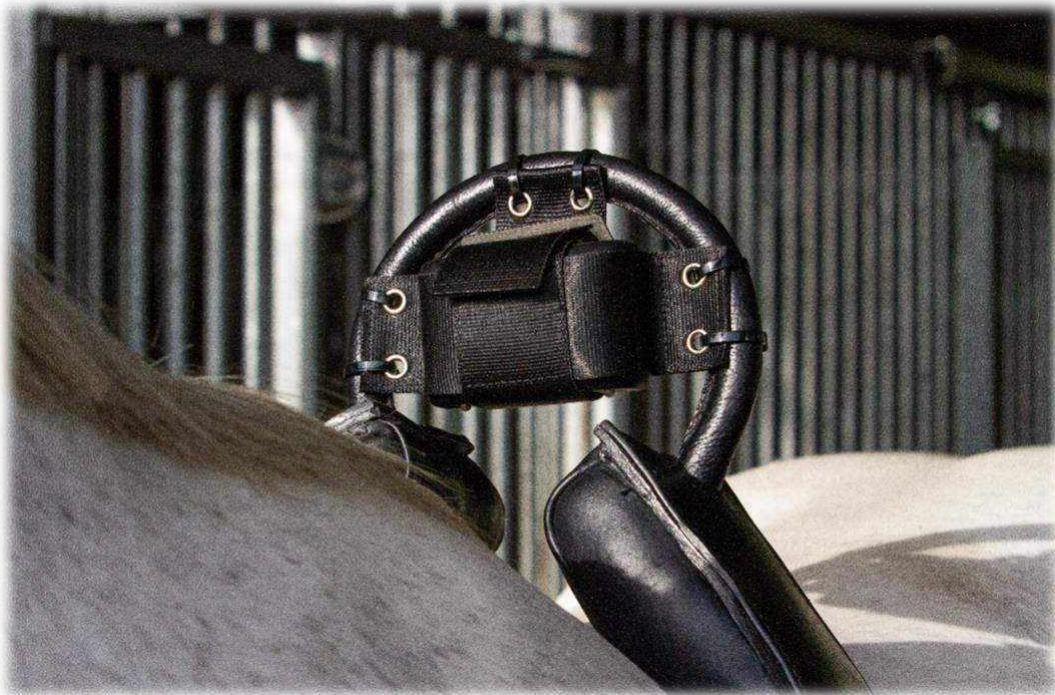

12 Summary and modes of use

Below is a brief description of how the Birth Alarm Premium works and how it should be used. We recommend that you read this text.

The Birth Alarm Premium checks the position of your mare once every 4 seconds. In the intervals the Birth Alarm Premium will switch to an energy-efficient state so that the battery is not overloaded.

When your mare has contractions and therefore lies down stretched, the Birth Alarm Premium will call your own mobile phone. You will then see the Birth Alarm Premium phone number appear on the screen of your mobile phone. If you save this number in the phone list of your mobile phone with for example the name "BirthAlarm" or "Mare", this name will also appear on your phone's screen. So now you know that the Birth Alarm Premium is calling you. Contrary to what you would normally do, in this case you **do not** answer your phone! As a result, you do not make a "call" and the notification to you will be completely free of charge. The Birth Alarm Premium stops ringing automatically after a few seconds. You can then call the Birth Alarm Premium back, which is also free of charge. With this you let the Birth Alarm Premium know that you have heard his call and the Birth Alarm Premium will stop calling you. The Birth Alarm Premium only becomes active again when the horse lies down again after some time.

If you do not call the Birth Alarm Premium back within half a minute, the Birth Alarm Premium will call the second number in its memory - if a second number has been programmed - and then wait another half minute for confirmation. For example, the Birth Alarm Premium can warn a total of 4 telephone numbers/persons in the event of a birth. If Birth Alarm Premium has called each number once without any of the numbers responding, Birth Alarm Premium will call the first number again. After this, the Birth Alarm Premium will also automatically send an SMS status message. This is an SMS message in which the Birth Alarm Premium has briefly and concisely stated some important information about the lying angle, battery capacity, network coverage, etc. **(see 9)**

Incidentally, it should be noted that the Birth Alarm Premium can also call a landline connection, which is your home phone, without any problems. If your landline is equipped with number recognition, you can also determine in advance whether the Birth Alarm Premium will call you. If your landline telephone has no number recognition, you will be obliged to answer every call. You will then hear a sound by which you can recognise the Birth Alarm Premium. In this case you do have a call connection with Birth Alarm Premium and a small amount will be deducted from the prepaid card of Birth Alarm Premium. If you would like to use Birth Alarm Premium in this way, we advise you to check your credit regularly. Make sure that this always remains above £10.

A set of alkaline batteries, will usually operate the system for at least a month. The use of rechargeable or the cheaper zinc-coal batteries is strongly discouraged!

13 Technical data

Supply voltage	4,5V (3xAA Alkaline)
Power consumption	200uA average (1,3A peak)
LTE Module	LTE FDD B1/B8/B7/B8/B20/B28 WCDMA B1/B8 GSM/EDGE B3/B8
Memories	4
Sleepmode	Yes
Detection angle	adjustable 65..80°
Notification types	Calling / SMS

Guarantee: Up to 2 years after date of purchase. Does not apply in the event of breakage, water damage and/or other damage. **The manufacturer and importer are not liable for any damage whatsoever arising from the use of the Birth Alarm Premium. Delivery and activities are carried out according to the general conditions of Gallagher Europe B.V.**

Gallagher Europe B.V. (NL) Bornholmstraat 62A 9723 AZ GRONINGEN Tel. 0031 (0)50 368 31 00 / Fax 0031 (0)50 306 16 55 E-mail: info.nl@gallaghereurope.com	
Gallagher Belgium b.v.b.a. (BE) Oude Baan 14 3945 HAM Tel. 013/66 77 66 / Fax 013/ 66 80 21 E-mail: info.be@gallaghereurope.com	
Gallagher Europe (DE) Talstrasse 7 49479 Ibbenbüren Telefon: 05451/9444-0 / Fax: 05451/9444-33 E-mail: info.de@gallaghereurope.com	

<p>Gallagher Europe (FR) Miniparc du Verger Bâtiment C - 1 rue de Terre Neuve 91940 LES ULIS Tél. 0820 203 700 / Fax 0820 203 701 E-mail: info.fr@gallaghereurope.com</p>	
<p>Gallagher Europe (AT) Hopfgarten 18 8741 Weisskirchen Tel. +43 (0)3577/82200 / Fax. +43 (0)3577/80921 E-mail: info.at@gallaghereurope.com</p>	
<p>Gallagher Schweiz AG (CH) Zürcherstrasse 499 9015 St. Gallen Tel. 071 / 313 29 50 / Fax. 071 / 313 29 60 E-mail: info@gallagherschweiz.ch</p>	
<p>Gallagher Powerfence UK Ltd. Curriers Close, Canley COVENTRY CV4 8AW Tel. 0844 8500101 / Fax 0844 8501545 E-mail: info.uk@gallaghereurope.com</p>	
<p>Gallagher Sweden AB (SE) Rönnaviksgatan 13 213 74 MALMÖ Tel. 040-41 88 80 / Fax 040-41 88 88 E-post: info@gallagher.se</p>	